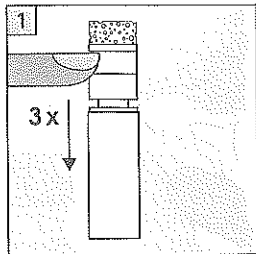
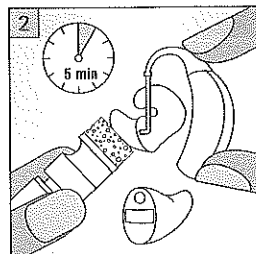
Cleansing stick
 long-term antimicrobial effects against bacteria, fungi, HBV- and HIV-viruses by moistening the entire surface. After application let dry for 5 min.

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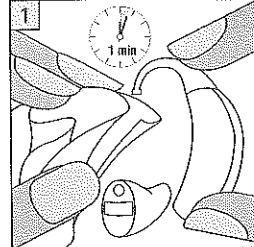
Cleansing Wipes
 Moistening the entire surface is effective against bacteria, fungi, HBV- and HIV-viruses. After application let dry for 1 min.

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Use disinfectants safely and cautiously.

ts, Inhaltsstoffe, Ingrédients, tes, Ingredienti, (100 g)
 anzalkonium chloride,
 yridin-2-thiol-1-oxide,
 ytrithion, polycondensates,
 is, purified water

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Drying capsules

Composition:
 silica with indicator
 (absolutely harmless)

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Attention!
 Not all products on this instruction might be included in your set!

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Phonak AG · Leuhisrütistr. 28
 8712 Stäfa · Switzerland
 www.phonak.com

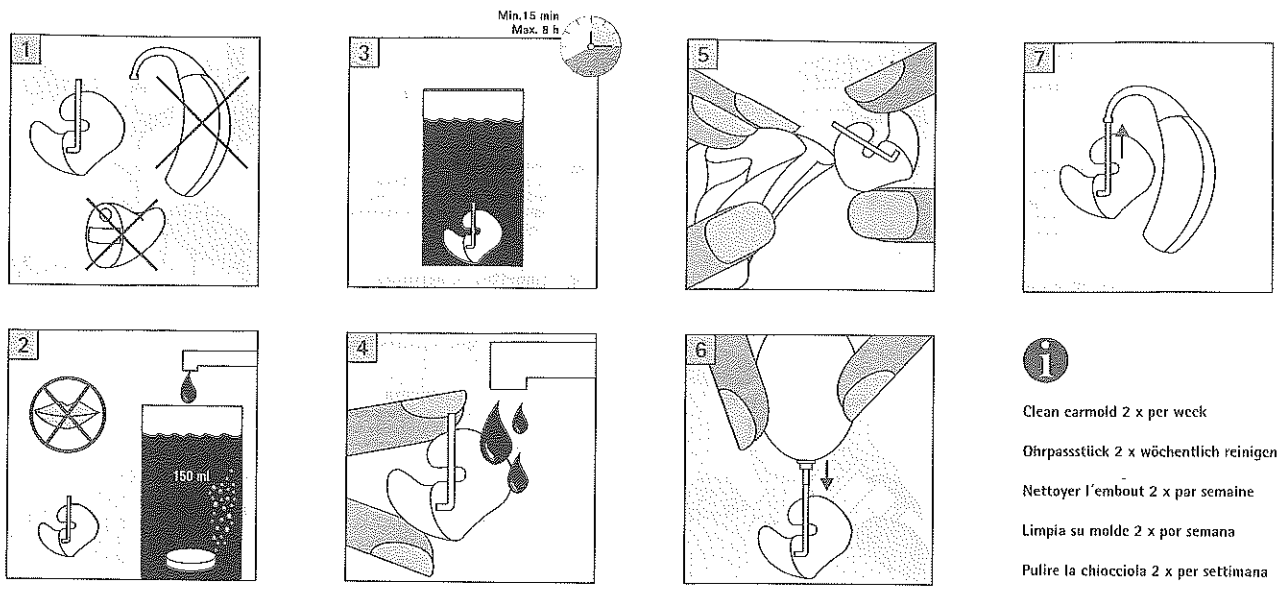
Cleansing
 Never put the hearing aid into the cleansing liquid!
Attention! Do not swallow the tablet!

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Drying
 Before drying, please remove the battery from your hearing aid. The capsule can be re-used until the golden yellow indicator has changed to a white color.
Attention! Capsule not to be opened! Do not swallow the capsule!

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Before drying, please remove the battery from your hearing aid. The capsule can be re-used until the golden yellow indicator has changed to a white color.
Attention! Capsule not to be opened! Do not swallow the capsule!

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